

Description

A one-piece threaded wing nut (D.I.S.S. 1240 oxygen fitting) and barbed nipple configuration. The groove bars help to secure the tubing and prevent oxygen leakage. The device is also referred to as a Christmas tree adapter. Available in a green (1205) or clear (1205C) color.

Intended Use



For use when connecting oxygen tubing to an oxygen source outlet, for example, flowmeters, cylinder regulators, concentrators. The Nipple & Nut adapter is a disposable, single patient, multiple-use device.

Material

Injected molded, Green ABS plastic. Not made with natural rubber latex.

Use Instruction

1. Remove the nipple & nut adapter from packaging.
2. Attach the nipple adaptor to oxygen gas source (flowmeter, oxygen regulator, oxygen concentrator).
3. Turn wing nut until tight.
4. Attach oxygen supply tubing to barbed nipple end.

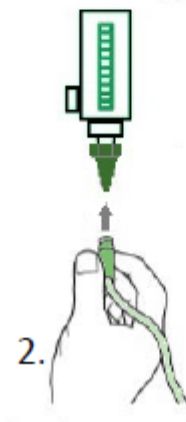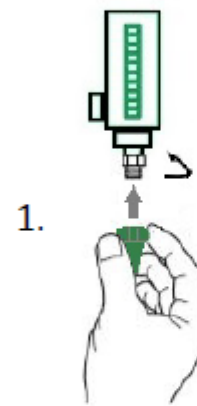

For the most recent revision of this technical bulletin and other product technical bulletins, visit Salter Labs website.

www.salterlabs.com/resources